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## User Instructions - Information for Users

<b>Product</b>	Sono E2 – Particle-filtering half mask FFP2 NR
<b>Manufacturer</b>	Dr. Freist Automotive GmbH, Alte Heerstr. 21, 38644 Goslar, Germany
<b>Declaration of Conformity</b>	A copy of the Declaration of Conformity for this model can be found online via the following link: <a href="http://www.dfa-gmbh.eu/Sono">http://www.dfa-gmbh.eu/Sono</a>
<b>Certification</b>	The masks fulfil all technical specifications of the PSA directive (EU) 2016/425 and bear the CE label. Certification according to module B, EU review, and module C2, EU quality control was issued by the TÜV NORD CERT GmbH, Langemarckstr. 20, 45141 Essen, Germany.

**Please read these instructions carefully before using the mask!**

### Inspection before Use

Check the mask for defects before application. The mask is no longer suitable for use if the use-by date is exceeded. An individual risk assessment must be done in particular cases.

### Application and Limits of Use

The particle-filtering half mask provides complete breathing protection and is predominately comprised of filter material. It can be used corresponding to class FFP2 for protection from solid and liquid aerosols. National provisions apply during use. Please become familiar with handling the product before use.

All materials used are odourless and free of harmful substances. The nose clip is nickel-free. The rubber band does not contain latex. To maintain protection never alter this mask.

The product is not sterile! The useful life of a mask should not extend beyond the length of one shift. Reuse is excluded. Please change the mask if defects are obvious, if there is increased breathing resistance or if it is moist. The product should be disposed of according to regional regulations.

The mask may not be used in poorly ventilated areas with an oxygen content below 17 vol.%. It affords no protection from solvent vapours or gases. Do not use in a potentially explosive atmosphere.

### Putting on the Mask

1. Please wash your hands for at least 20 s with soap or disinfectant to ensure adequate protection.
2. Glasses should be removed before putting on the mask.
3. Hold the mask by the bands or at the edge and check these for defects. Please use another mask if there are defects.
4. Fold the mask into a cup and thereby take note of the nose clip on the upper side. Hold the bands tightly, hold the mask in front of your face and pull the bands behind your ears. Adjust the nose bracket so that it seats to fit the bridge of the nose.
5. Check the tight fit of the mask's seal by blowing out vigorously. If air escapes at the sides or at the nose, correct the mask's fit accuracy and refit the nose clip.

The mask provides adequate protection only when a secure seal has been achieved. Facial hair and scars can impair proper mask seating.

### Storage

Store the mask at room temperature in a dry, well ventilated place. Keep away from sources of fire. Max. storage time: 36 months from the date of production. Expiration date, see packaging and mask.

 Storable until  10°C optimumstorage temperature  max. rel. humidity Storage

For all other information please contact the manufacturer. Dr. Freist Automotive GmbH - Alte Heerstr. 21 - 38644 Goslar - Germany - Website: [www.dfa-gmbh.eu](http://www.dfa-gmbh.eu) - Email: [info@dfa-gmbh.eu](mailto:info@dfa-gmbh.eu)

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